

# Use of a CeraPlus™ Skin Barrier with Remois Technology\*

## Case Study 5

### Abstract

There is a high incidence of peristomal skin complications, with more than half of all people living with an ostomy experiencing a peristomal skin issue at some point in their lifetime<sup>1</sup>. The types of complications, the reasons for them, and the solutions used to treat them can vary widely. For clinicians, managing these peristomal skin complications takes time and effort. For patients, sore peristomal skin can have a huge impact on their quality of life. Peristomal skin complications are the most common post-operative complication following creation of a stoma<sup>2</sup>.

### Aim

To visibly improve and maintain peristomal skin integrity by finding a suitable skin barrier formulation for the patient, and ensuring a proper skin barrier fit around the stoma.

### Patient Overview

The patient is a female adult with Systemic Lupus Erythematosus (SLE). Systemic lupus erythematosus, often abbreviated as SLE or lupus, is a systemic autoimmune disease in which the body's immune system mistakenly attacks healthy tissue<sup>3</sup>.

The patient lives with her adult son in a ground floor flat as she is wheelchair bound. The patient had a cystectomy and formation of a urostomy for bladder cancer. The patient had been using the same two-piece extended wear pre-sized skin barrier for three years with a wear time of two-three days, before she developed peristomal skin complications.

### Problem

The patient contacted the stoma care nurse to say the two-piece skin barrier she was using would no longer adhere to her skin and her peristomal skin had become sore, wet and itchy (**Photo 1**). She was experiencing leaks as the skin barrier would not stick to her skin, which in turn was exacerbating the skin complaint. Wear time had significantly reduced to only a few hours.

Initially a skin swab was taken. The result of this was inconclusive. As she suffered with systemic lupus erythematosus (SLE) it was thought that her skin complaint could be a manifestation of her lupus, known as acute cutaneous lupus, as she was also experiencing kidney problems due to a lupus "flare-up".

Some people with lupus can develop a type of skin disease called acute cutaneous lupus. The most typical form presents as a "butterfly rash" or flattened areas of red skin on the face. However, this skin complication can also appear on other areas of the body. The rash does not typically produce scarring, although changes in skin color may occur<sup>4</sup>.

### Interventions

After unsuccessfully trying different barrier preparations on the peristomal skin, the decision was made to use the two-piece **CeraPlus** cut-to-fit flat skin barrier.

*continued on back*



**Photo 1** Peristomal skin is moist with areas of redness. Patient complained of itching and soreness.



**Photo 2** Improvement seen to the peristomal skin after one week.



**Photo 3** Continued improvement seen to the peristomal skin after using the CeraPlus skin barrier for three weeks.

# Use of a CeraPlus Skin Barrier with Remois Technology\*

Contributing Author and Affiliations

**Louise Foulds RGN**  
Clinical Nurse Specialist in Stoma Care  
Hollister Limited

## Case Study 5

### Outcomes

Improvement was seen to the peristomal skin after one week. The peristomal skin was drier and the patient reported a reduction in itching and soreness (**Photo 2**). Continued improvement to the peristomal skin was noted after three weeks. At this time, the patient no longer complained of itching or soreness (**Photo 3**).

### Conclusion

Use of a CeraPlus skin barrier as part of the overall plan of care for this patient provided a positive outcome. She felt a sense of confidence and reassurance due to the adhesion and increased wear time. The peristomal skin returned to normal condition and the symptoms of pain and itching were eliminated.

Many people with ostomies experience peristomal skin issues and accept them as a normal aspect of having a stoma<sup>5</sup>. Thankfully, this patient sought help and there was visible improvement of the peristomal skin. Achieving a good fit around the stoma and preventing leakage as a means of mitigating skin irritation may not be enough to keep the peristomal skin healthy. The formulation of a skin barrier also has an impact on the health of the peristomal skin. Finding the right combination of skin barrier formulation, and skin barrier fit is essential to maintaining a healthy peristomal skin environment.

### References:

1. Richbourg L, Thorpe J, Rapp C. *Difficulties experienced by the ostomate after hospital discharge*. J Wound Ostomy Continence Nurs. 2007; 34(1):70.
2. Meisner S, Lehur P-A, Moran B, Martins L, Jemec GBE. *Peristomal Skin Complications Are Common, Expensive, and Difficult to Manage: A Population Based Cost Modeling Study*. PLoS ONE. 2012; 7(5): e37813.
3. "Systemic lupus erythematosus." In *Wikipedia, The Free Encyclopedia*, 14 Aug. 2015, [https://en.wikipedia.org/w/index.php?title=Systemic\\_lupus\\_erythematosus&oldid=734497167](https://en.wikipedia.org/w/index.php?title=Systemic_lupus_erythematosus&oldid=734497167). Accessed 9 July 2016.
4. Sontheimer, Richard D. "How Does Lupus Affect the Skin?" Lupus Foundation of America, Inc., 12 July 2013, <http://www.lupus.org/answers/entry/how-lupus-affects-skin>. Accessed 9 July 2016.
5. Whiteley IA and Sinclair G A *Review of Peristomal Skin Complications Following the Formation of an Ileostomy, Colectomy or Ileal Conduit*. World council of Enterostomal Therapists Journal, 2010; 30(3) p. 23-29.

*The support of Hollister Incorporated is gratefully acknowledged.*

*This case study represents one nurse's experience in using a two-piece CeraPlus cut-to-fit flat skin barrier with a specific patient and may not necessarily be replicated.*



\*Remois is a technology of Alcare Co., Ltd.

Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions.



Not all products are CE marked.

The Hollister logo and CeraPlus are trademarks of Hollister Incorporated. All other trademarks and copyrights are the property of their respective owners. © 2016 Hollister Incorporated



**Hollister Incorporated**  
2000 Hollister Drive  
Libertyville, Illinois 60048 USA  
1.800.323.4060

[www.hollister.com](http://www.hollister.com)